E

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GB

Immobilisation is a basic principle for treatment of injuries and overstrains, as recovery process will be considerably accelerated and consequential damages prevented.

Shoulder arm supports are relieving affected tissue structures (joints, ligaments, capsules, cartilages).

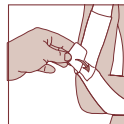
With the BORT OmoBasic® Shoulder Support acc. to Gilchrist shoulder joint and elbow can be rested in adduction position to the body in a very comfortable and safe way, thus wound inspection without problems is provided.

Instructions for use:

The BORT OmoBasic® acc. to Gilchrist should be put on by a specialist.

1. The support should be put on with the elbow bent by 90°. Lead the hand through the hand loop, which can be adjusted to the correct width by means of a Velcro closure.

2. Now put on the forearm rest, so that the forearm is positioned in the support, and the blue straps surround laterally the elbow joint and rest against the upper arm. The width of the support at the upper arm can be adjusted by means of a Velcro closure.

3. As the next step position the shoulder part into height of shoulder and lead the sewed-on shoulder strap around the neck to the front.

4. In order to stabilise the forearm, lead through the shoulder strap below the wrist, turn it around upwards and close it with the Velcro closure.

5. Finally lead the body strap, which is fastened at the hand section, around the back towards the arm, which shall be immobilised, and there between the upper arm and body again to the front side and fix the end by the Velcro closure onto the body strap positioned at the back.

If required the hand can additionally be fixed with the attached narrow, black strap.

Indications:

Sprains, contusion, injuries to the rotator cuff, dislocation, subcapital humerus fractures, fractures of the humerus ball and shaft, after surgical intervention.

Contraindications:

Large-area skin injuries, large-area edemas, congestion in the lymphatic system, Artery Occlusive Disease (AOD), oversensitivity against one of the materials used.

Composition:

43% Polyamide, 33% PUR foam, 24% polyester

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

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