
**COMBIFLEX®  
VISCOFLEX®**

© SYSTEMAM® 062007 - Toute reproduction interdite: 1569D/6 - Réf: 76000326

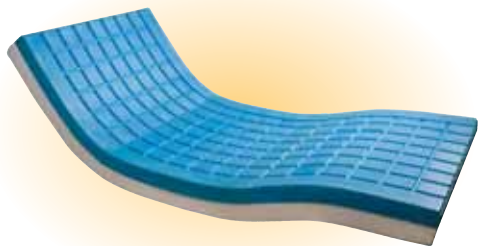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**GB** The first preventive measure against bedsores is to change the supporting points of the body on the mattress, on average every three hours, when in bed. When in a sitting position on a cushion, the body should be raised as frequently as possible to limit compression of the blood vessels, thereby enabling a correct circulation of the blood.

**STORAGE**

- Do not expose to direct high heat.
- For good longevity, it must be stored in a warm dry place, away from heat and light.

**MAINTENANCE / CLEANING**

**DO NOT IRON. DO NOT USE CORROSIVE PRODUCTS OR ABRASSIVE MATERIALS.**

Wash in antiseptic soapy water, rinse in clear water. Dry in open air. Hand wash only. Do not use bleach. Do not iron. Do not use a tumble dryer.

**DISINFECTING / DECONTAMINATING**

Use cold spray products (ammonium quaternaire-glutaraldéhydes, etc.).

**ENVIRONMENT**

- Do not burn or dump in the countryside.

**SAFETY**

VISCOFLEX® - POLYMAILLE® complies with the fire safety regulations in force.

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*Mattress composed on the surface of Polyurethane, viscoelastic shape retaining memory foam of high density, moulded into a base (with inserted anti-punching node) of polyurethane high resilient, high density foam. POLYMAILLE® protective cover. Mattress composed of three identical elements.*

**WARNINGS & PRECAUTIONS :**

For reasons of hygiene, all medical devices used for the prevention of bedsores should be reserved for the exclusive use of one patient.

In the event of incontinence, the cover and eventually the mattress should be cleaned. It is advisable to purchase a second protective cover for use while the first is being washed, so as to avoid direct contact with the surface of the mattress.

Do not place any other materials or objects between the mattress and the patient's clothing, apart from the protective cover and a bed sheet.

**Fig. 1.**

- **A.** Minimum weight that can be supported by the mattress without reducing its efficiency :

- **B.** Maximum weight that can be supported by the mattress without reducing its efficiency :

The ideal temperature range for use is between 15°C and 45°C, (49°F and 122°F).

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