- (EN)** Z1 Mobile App User Guide
page 2
- (DA)** Brugervejledning til Z1 Mobile App
page 8
- (DE)** Z1 Handy-App, Gebrauchsanweisung
page 13
- (ES)** Guía del usuario de la aplicación para móviles Z1
page 19
- (FI)** Z1-mobiilisovelluksen käyttöopas
page 25
- (FR)** Guide de l'utilisateur de l'application Z1
page 31
- (IT)** Guida per l'utente di Mobile App Z1
page 37
- (NL)** Gebruikersgids Z1 Mobile App
page 43
- (NO)** Bruksanvisning for Z1 mobilapp
page 49
- (PL)** Instrukcja obsługi aplikacji do aparatu Z1 na urządzenia mobilne
page 55
- (PT)** Guia do utilizador da App Z1
page 61
- (RU)** Руководство пользователя мобильного приложения Z1
page 67
- (SV)** Bruksanvisning för mobilappen för Z1
page 73
- (TR)** Z1 Mobil Uygulaması Kullanıcı Kılavuzu
page 78

BREAS

Manufacturer:
Breas Medical AB
Företagsvägen 1
SE-435 33 Mölnlycke
SWEDEN
Phone: +46 31 86 88 00
Fax: +46 31 86 88 10
E-mail: breas@breas.com
Web: www.breas.com

CE 0123

EN User Guide

Intended Use

The Z1 Mobile App can be used with the Z1 Auto CPAP device to control settings and view treatment data.

The Z1 Mobile App can be used to control CPAP and APAP device features on your Z1 device.

The Z1 Mobile App can only be installed on iPhone 4S and beyond.

Connect to the Z1

1 Make sure that Bluetooth is activated on your Z1 device.


You activate Bluetooth by holding the Up Arrow Button immediately after having plugged the AC adapter to the Z1 device.

2 Open the Z1 App on your phone and then press the “+” button to connect to your Z1 device.

3 Select your Z1 device.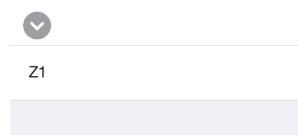


The first time you connect to your Z1 you need to enter the Z1 device Bluetooth password.

The default password is “1234567”.

You can also change the Bluetooth password for your Z1 device by tapping “Update password”.

You can at any time change the password in the Options menu (⚙️).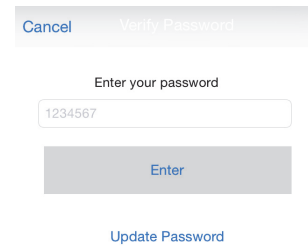

4 The App is now connected to your Z1.

Remote Control


When connected you can use the App to remote control your Z1 by using the buttons in the App the same way as you do on your Z1 device.

See your Z1 User Guide for how to control your Z1.

Setting Treatment Parameters

1 Swipe left from the start screen to access the Settings screen.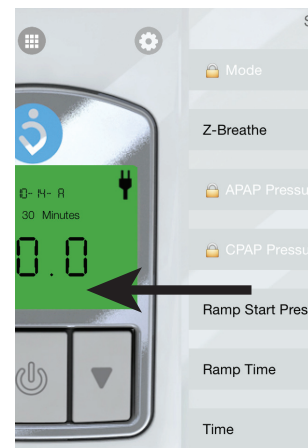


2 Press a parameter to change the setting.

3 Set the value:

Mode and Z-Breathe

Press the desired value to set it.

Then press “Save”.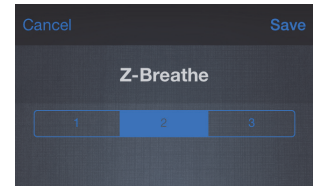

Pressure and Ramp Time

Drag the slider to set the value.

Then press “Save”.


Set Time

1 Swipe left from the start screen to access the Settings screen.


2 Select “Time”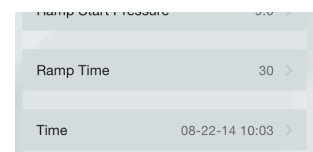

3 Adjust the values using the “-” and “+” buttons. You can also use the phone’s time by pressing “use current time”.

Then press “Save”.

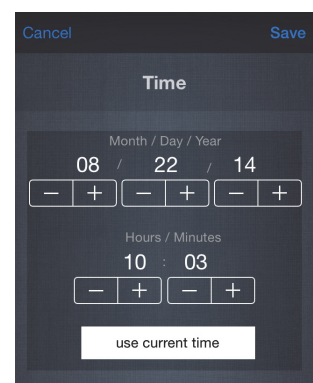
View Treatment Data

Press the  icon to access the treatment data view.

Sync the treatment data with the connected Z1 by pressing the  icon.

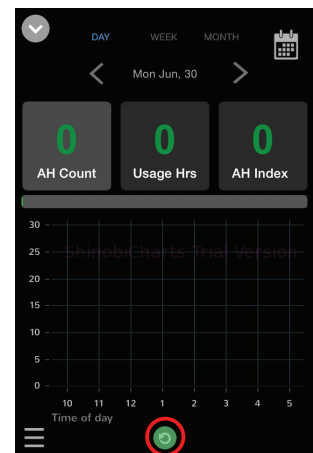
Select time frame

You can view the treatment data in three different time frames: day, week or month. Press the text at the top of the screen to select time frame.

You can also switch time frame by zooming in or out in the graph.

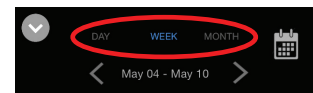- Zoom in by tapping the desired part of the graph you want to study closer.
- Zoom out by using the two fingered pinch-in gesture.


Select time period

There are several ways to select which time period you will view:

- Use the calendar icon  and select the desired date.
- Use the  and  icons to go to the previous or next time period.
- Swipe left or right to go to the previous or next time period.

Switch between AH Index and Pressure

You can switch the rightmost data value in the data view screen between “AH Index” and “Pressure” display by double-tapping the value.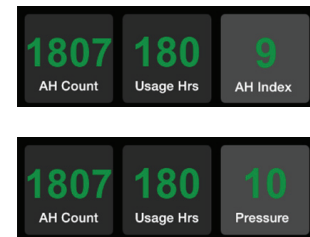

Displayed Treatment Data Parameters

AH Count:

The number of Apopnea/Hypopnea events that has occurred during the selected time period.

Usage Hrs:

How many hours the Z1 has been used during the selected time period.

AH Index:

The number of Apopnea/Hypopnea events per hour during the selected time period.

Pressure:


The average pressure during the selected time period.

Options

Press the  icon to access the options screen.

In the options section you can change settings for the data view graphs, change the Bluetooth password and manage App users.

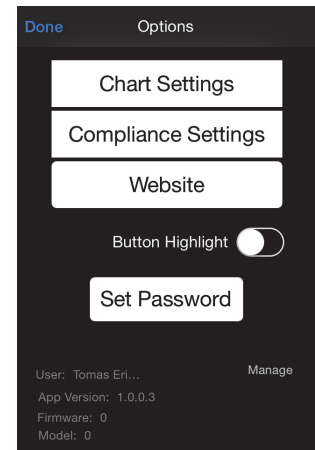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Z1 Mobile App kan bruges sammen med Z1 Auto CPAP-apparatet til at styre indstillingerne og få vist behandlingsdata.

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