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GB

The BORT Stoma Support provides support in case of abdominal tissue infirmity especially also in the area of artificial intestine exits. In the anus praeter area the pressure pad provides for stabilisation of the affected tissue structures.

Instructions for use:

For the application with pad, first place this by Velcro fixing at the inner face of the support in the area of the stoma opening. Put on support loosely. Open both Velcro closures of the stoma opening, position open-ing around the stoma area. Pull stoma bag through the opening and fix both Velcro closures again. Finally close the support in a way, that suf-ficient and comfortable stabilisation effect is obtained.

Indications:

Condition after ileostomy and colostomy with accompanying weakness of connective tissue.

Contraindications:

Large-area skin injuries, lymphedema, septic phlebitis, oversensitivity against one of the materials used.

Composition:

55% PES foam, 25% polyamide, 15% cotton, 5% spandex

Important instructions:

No reutilisation – this medical device is meant for the care of one pa-tient only.
Wear support only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen support and re-move it if necessary. If complaints continue, please contact the doctor.

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